



GetMeKnown PTY Ltd.

PO Box 1081; Malmesbury; Cape; 7299

“For Service Excellence”



Microchips and Scanners adhering to the ISO FDX-B (15digits), 11784/85 standards for International recognition.

About Us:

The Goal of GetMeKnown is to create an Online Microchip Management Service that will allow you to manage all your registered Micro-Chips in an easy to understand manner while still giving you complete control.

With features that allow you to transfer ownership that isn't time consuming as well as logging your actions for future viewing. That means you would have documented records at your fingertips.

You're in Good Hands and here's why;

- Get Me Known adheres to the International standards like ISO11784/5 FDX-B, ICAR, etc. Read by all ISO animal scanners.
- Backed by a Company in existence and supporting animals for the past 7 years with various other products.
- International Accessible Database stored on rack servers - We have never been offline from public launch. Truly a 24/7 operation.
- Innovation of a breeder to assist the breeders, shelters, Vets, approved **for Import and Export, Etc.** with no hidden cost.
- Extra features like to flag a missing animal, Record card for shelter management, etc..
- No Annual Fees !!!!!!!!!!! Pay once only.

Our Goal :

To make micro chipping affordable whilst providing high quality products to allow each responsible owner, Vet or Shelter to microchip and register their pets/animals online and “get them known” when lost or resolve queries regarding legal ownership

Register a New Account

To perform any account related action on **GetMeKnown** (<http://www.getmeknown.co.za/>) you'll need an account, and creating one is quite easy. To start head over to <http://www.getmeknown.co.za/account/register> and start the process. Reminder that you are the injector and to use your own id no. or your KUSA/Other membership number.

Register a Microchip

After you have created a account you will need to Login (<http://www.getmeknown.co.za/account/login>), this is done by providing the E-Mail Address you used to register and the password for your account. To create a Microchip click on "**Register Microchip**" and follow the directions on the next screen. You will then be asked to provide your Microchip Number. A Microchip Numbers consists of 15 Digits. Please verify correctness. Complete all the required fields. To view all your registered pets just click “account”.

Print or e-mail Certificates

Once you have completed a registration of a microchip you can view, print or e-mail the certificate. Should you need a copy at a later stage just login to your account and press “Certificate” next to the desired microchip number.

Transfer ownership of a microchip

To Receive Ownership of a Microchip you will need to Make a Request and wait for the current owner to login and then approve your request. **No need to type in any information about the Pet.** Login to your account, then in the Main Menu on your profile page select "Request Ownership". Type in the 15 digit microchip number. You will be redirected to a page where you can review the details of the pet click "Request a Transfer of Ownership" the request will be sent you just need to wait for the current owner to accept.






GetMeKnown PTY Ltd.

PO Box 1081; Malmesbury; Cape; 7299

“For Service Excellence”



Microchips and Scanners adhering to the ISO FDX-B (15digits), 11784/85 standards for International recognition.

<p>RFID (microchip) Concept: This is not a GPS microchip and cannot be tracked via satellite. It is a permanent identification chip that is implanted under the animals’ skin between the shoulder blades and must be scanned with a microchip reader/scanner at close range (max +- 12cm). You must register the microchip online and then when the database is searched you will be identified as the owner. Our vision is that every Animal is allowed an unique identification.</p> 	<p>Our Products: Microchips: Our Microchips adhere to the ISO Standard 11784/85 FDX-B 15 digit international standards and is also ICAR recorded (International 15 digit Microchips).</p> <p>Universal Reader: The Lusahn Halo RFID Reader are able to read all the 15 digit (FDX-B), older 10 digit ID codes (FDX-A) and even the Bio-Thermo Microchips. Imported from the UK where it is extensively used by Vets, Police, shelters, breeders, etc. All microchips can be read.</p>		
<p>Our Database: International Accessible database stored on rack servers - We have never been offline from public launch. Our Infrastructure is Maintained by Certified Engineers. No paperwork as you manage all your microchips online, you can even e-mail certificate to all relevant people. Accepted by KUSA, CASA and pet medical aids etc.</p>	<p>Contact Us: Website: www.getmeknown.co.za Office: 022 485 7310 Cell: 082 373 6375 Fax: 0880224857310 fax to email info@getmeknown.co.za Contact : Adri/Johan</p>		
Microchip			
1 – 4	R70.00	20 – 49	R55.00
5 – 9	R65.00	50 – 99	R50.00
10 – 19	R60.00	100 – more	R45.00
NPO Rescues Only >100 order		R40.00	
Halo Scanner FDX (for all international 15 and 10 Digits Microchips/Transponders)			R1780

Unique features :

- You as account holder/ owner may flag the microchip to display a online message when dog was stolen, lost, etc..
- Stolen animals can be “hotlisted” with the information that is uploaded to all Halo scanners and their details will display on the screen when scanned.

